

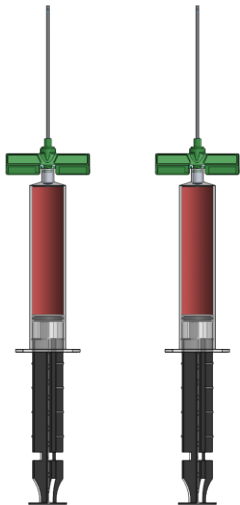
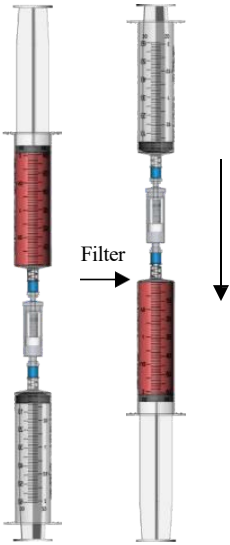
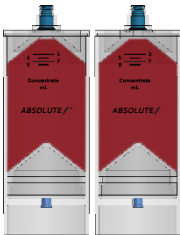
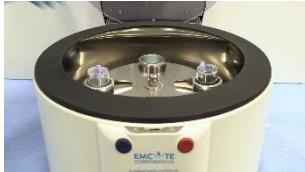
## GSBMA-120: IFU ILLUSTRATION

**NOTICES: PLEASE DISCARD RED VENTED CAP FROM CONCENTRATING DEVICE BEFORE USE.  
ALWAYS SWAB SELF-SEALING PORT WITH STERILE ALCOHOL PRIOR TO ACCESSING WITH A STERILE SYRINGE.**

### PREPARATION PROTOCOL:

|                                                                                                                                                                                                                                                  |                                                                                                                                                                                                     |                                                                                                                                                                                                                         |                                                                                                                                                                                       |
|--------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|-----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|-------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|---------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| <p><b>STEP 1:</b></p>  <p>Attach the sterile filter needle onto the VACLOK 60mL syringe. Then draw 15mL of Heparin Anticoagulant<br/><b>(1000 units/mL)</b></p> | <p><b>STEP 2:</b></p>  <p>Then prime the bone marrow aspirating cannula by injecting 5mL of heparin through it</p> | <p><b>STEP 3:</b></p>  <p>Attach to the OUT port of the bone marrow filter. Fill to prime and then aspirate back into the syringe</p> | <p><b>STEP 4:</b></p>  <p>Then discard the residual heparin leaving 5mL in each VACLOK syringe</p> |
|--------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|-----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|-------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|---------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|

### CONCENTRATING PROTOCOL

|                                                                                                                                                                                                                                                          |                                                                                                                                                                                                                        |                                                                                                                                                                                                          |                                                                                                                                                                                                                                                                                                                                                                    |
|----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|--------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| <p><b>Step 1:</b></p>  <p>Slowly draw 55mL of bone marrow aspirate, filling each syringe to 60mL. Mix the BMA and heparin upon collection to prevent coagulation.</p> | <p><b>Step 2:</b></p>  <p><b>Start End</b></p> <p>For each BMA syringe filter the BMA in the direction indicated on the filter.</p> | <p><b>Step 3:</b></p>  <p>Inject 60mL of anticoagulated &amp; filtered BMA into each <b>Concentrating Device</b></p> | <p><b>Step 4:</b></p>  <p>Counterbalance with the same volume in each Concentrating Device.</p>  <p>Then place them directly opposite to each other in the centrifuge rotor buckets.</p> |
|----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|--------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|


**STEP 5:**

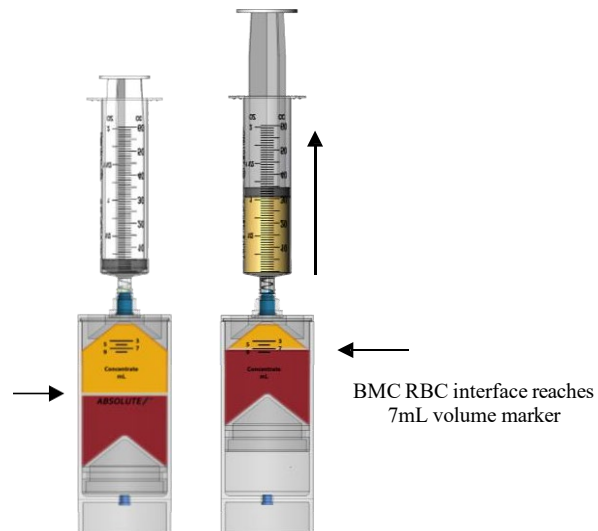Sapphire Series  
Centrifuge:  
**AbsolutePRP/BMC**

Platinum Series  
Centrifuge:  
**AbsolutePRP/BMC**


Executive Series  
Centrifuge:  
**10 minutes**  
**4400 RPM**

**STEP 6:**



Attach the 60mL syringe and aspirate the plasma until the bone marrow RBC interface reaches the 7mL volume marker, then stop aspirating

**REPEAT FOR EACH CONCENTRATING DEVICE:**

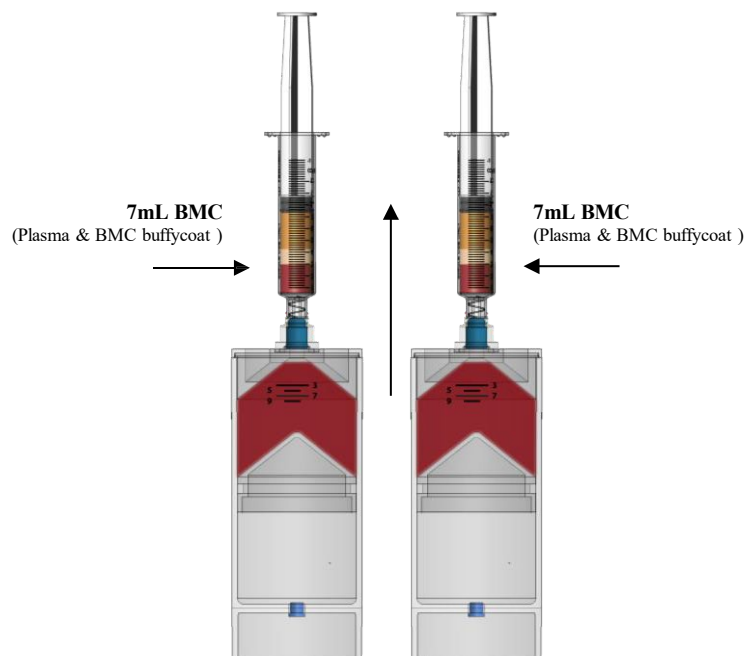

**FOR EACH DEVICE**

Attach the 12mL syringe and aspirate 7mL BMC  
(Rotate the syringe to re-suspend the BMC buffycoat into the plasma)

**COLLECT A TOTAL OF 14mL BMC**